

Quick Reference Guide for Cleaning/Sanitizing Secure Care Products Equipment

COVID-19 | Updated March 30, 2020



Cleaning Resident/Patient Transmitters

- We recommend cleaning transmitters after each use, typically with a lint-free isopropyl alcohol (IPA) wipe.
- In the absence of such wipes, other disinfectants can be used as an alternative, provided they are safe for use on plastic surfaces.
 - Recommended cleaners:
 - *MediChoice brand medium sized alcohol wipes (using larger wipes could cause false alarms due to excess moisture.)*
 - *Spartan Chemicals Super HDQ may be safely used on Secure Care Products' equipment*
- With the transmitter removed from the resident or patient, wipe the transmitter on all sides with the disinfectant.
- Ensure there are no lint fibers attached to the transmitter after cleaning



Cleaning Transmitter Strap Latching Mechanism

- Fold the prep pad in half lengthwise and insert upward in the same direction as the strap gets inserted

For Standard Transmitters

- Grab the pad on both ends and gently slide it back and forth along the width of the latching mechanism

For Cutband Style Transmitters

- DO NOT slide the pad back and forth, only pull the wipe through in one direction (repeat as needed)

It is important to be careful not to get the wipe snagged and cause bending/deforming of the metal contacts inside this latching area

- Once clean, place in a containment box

Always store transmitters in a transmitter containment area when not being used on residents or patients.



Cleaning Exit Panels and Keypads

Analog Push-Button and Legacy Exit Panels

- May be wiped with typical disinfectant/antibacterial wipes
- We do not recommend spraying cleaners directly onto the exit panels
 - Instead, spray onto a cloth or paper towel and wipe surfaces as needed

Touchscreen Exit Panels (Evolution Series)

- The plastic housing/bezel may be wiped with typical disinfectant/antibacterial wipes
- Touchscreen must be cleaned with products that do not use harsh chemicals like ammonia, or alcohol-based cleaners
 - Klear and iKlenz are recommended commercial cleaners
 - If preferred, a 50/50 mix of vinegar and water can also be used
- Use a soft lint-free cloth (*avoid abrasive cloths, paper towels or tissues paper which may damage the touchscreen*)



Questions or Concerns?

**Locate and contact your local
Secure Care Products certified distribution partner**

or

Contact us directly

info@securecare.com | +1 (603) 223-0745 | www.securecare.com

